


Coverall suit - instructions for use

PUTTING SUIT ON

- 1 Remove footwear
- 2 Put feet in first
- 3 Pull suit up and put arms in sleeves
- 4 Close front zipper
- 5 Secure flap with tape over closed zip
- 6 Position hood over head
- 7 Put footwear on

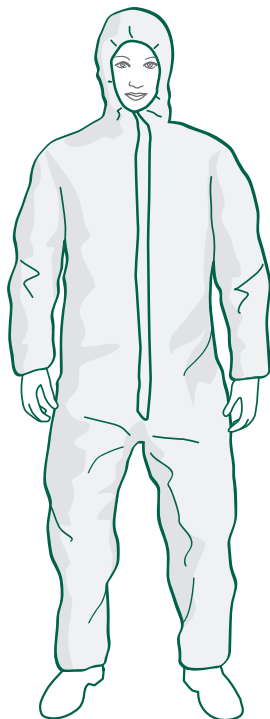

TAKING SUIT OFF

- 1 Remove footwear
- 2 Lower the hood
- 3 Open zipper
- 4 Undress arms
- 5 Undress legs
- 6 Put footwear on
- 7 Dispose of suit as instructed by your organisation's waste management protocols

Specifications

- Coverall suit
- Size XL
- Non-woven fabric
- White in colour
- Overlocked seams
- Zip with cover
- Elasticated cuffs, ankles and hood
- Handmade in Yorkshire, England



Suits are not certified and are worn at the user's discretion

Product Code
ORV+DW10N+01



Don't forget to recycle packaging.

Unless clinically soiled, all products can be recycled
WARNING: Please dispose of plastic bag safely - it can cause suffocation



✓ Store out of direct sunlight in a cool dry room indefinitely.

