


Gown - instructions for use

PUTTING GOWN ON

- 1 Fully cover torso neck to knees
- 2 Insert arms to end above the wrist
- 3 Wrap gown around the back
- 4 Fasten shoulder line tie
- 5 Fasten waist line tie

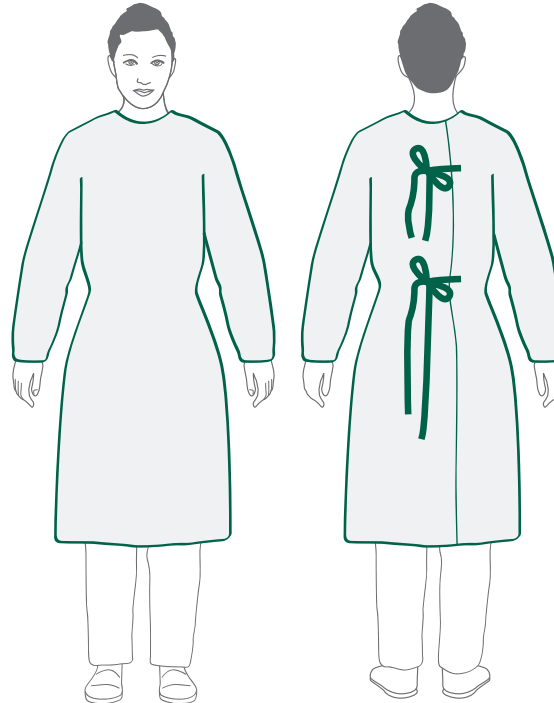

TAKING GOWN OFF

- 1 Unfasten waist line tie
- 2 Unfasten shoulder line tie
- 3 Undress arms
- 4 Dispose of gown as instructed by your organisation's waste management protocols

Specifications

- Gown
- Size XL
- Non-woven fabric
- White in colour
- Overlocked seams
- Elasticated cuffs
- Shoulder tie
- Waist tie
- Generous wrap on the reverse
- Handmade in Yorkshire, England



Gowns are not certified and are worn at the user's discretion

Product Code
ORV+DW10N+02



Don't forget to recycle packaging.

Unless clinically soiled, all products can be recycled
WARNING: Please dispose of plastic bag safely - it can cause suffocation



✓ Store out of direct sunlight in a cool dry room indefinitely.

