

Re-Usable Face Mask

PUTTING MASK ON

- 1 Open mask fully
- 2 Place the mask over the nose and mouth
- 3 Place elastic straps behind the ears

TAKING MASK OFF

- 1 Do not touch the front of the mask
- 2 Remove by unhooking the straps from behind the ears
- 3 Launder at 60°C in a domestic washing machine
- 4 Wash your hands



Specifications:

- Re-usable face mask
- Washable
- 100% filament polyester
- Fabric and thread silicone free
- Length of face mask – 15cm (+/- 10%)
- Width of face mask – 23cm (+/-10%)
- Elastic ear loops – 20cm (+/- 10%)
- Hemmed top and bottom
- Colour – dark blue
- Country of origin – El Salvador

General Use Mask – Not for virologic or bacterial protection

500 pieces per case

Single bag liner

**Product Code
ORV+MA10N+02**



Don't forget to recycle packaging.

Unless clinically soiled, all products can be recycled
WARNING: Please dispose of plastic bag safely - it can cause suffocation



✓ Store in a cool dry area away from natural sunlight and off the floor

