

## Re-Usable Face Mask


### PUTTING MASK ON

- 1 Open mask fully
- 2 Place the mask over the nose and mouth
- 3 Place top strap around head and securely fasten in a double bow
- 4 Place bottom strap around head and securely fasten in a double bow



### TAKING MASK OFF

- 1 Do not touch the front of the mask
- 2 Remove by undoing bottom strap first then top strap
- 3 Launder at 60°C in a domestic washing machine
- 4 Wash your hands

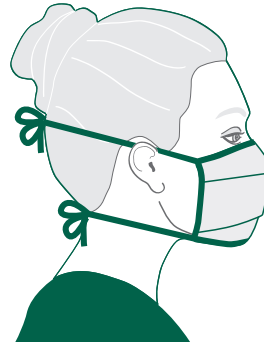

#### Specifications:

- Re-usable face mask
- Washable
- 100% filament polyester
- Fabric and thread silicone free
- Length of face mask – 23cm (+/- 10%)
- Width of face mask – 8cm (+/- 10%)
- Fabric tie length – 43cm (+/- 10%)
- Bound top and bottom extended to form ties and bound on the edges too
- Colour – dark blue
- Country of origin – El Salvador

**General Use Mask – Not for virologic or bacterial protection**

**500 pieces per case**

**Single bag liner**

**Product Code  
ORV+MA10N+03**



#### Don't forget to recycle packaging.

Unless clinically soiled, all products can be recycled  
**WARNING:** Please dispose of plastic bag safely - it can cause suffocation



✓ Store in a cool dry area away from natural sunlight and off the floor

